Freka® EasyIn , ENFit

REF 7981833

LOT XXXXXXXXX

Name: _____

Date: _____

Fresenius Kabi AG

7981833-PL-00

Freka EasyIn, ENFit

REF 7981833

270cm 2,6mm 1,8mm

YYYY-MM-DD

YYYY-MM-DD

LOT XXXXXXXX

PZN: 11345475

(01)04250273726527(17)YYMMDD(10)XXXXXXXX

- de - **Doppellumige Sonde**
zur intestinalen Ernährung (CH 8) gastralen Dekompression (CH 16). **Nur zur enteralen Anwendung.**
- en - **Double lumen tube**
for intestinal feeding (FR 8) gastric decompression (FR 16). **For enteral use only.**
- fr - **Sonde double lumière**
pour alimentation intestinale (CH 8) régulation de pression gastrique (CH 16). **Uniquement par voie entérale.**
- bg - **Двойно луменна сонда**
за интестинално хранене (CH 8) гастрална декомпресия (CH 16). **Само за ентeрално приложение.**
- cs - **Sonda s dvojitou**
dutinou k intestinální výživě (CH 8) ke gastrické dekompresi (CH 16). **Pouze pro enterální použití.**
- da - **Dobbeltlumen sonde til**
intestinal ernæring (CH8) gastrisk dekompression (CH16) **Kun til enteral anvendelse.**
- el - **Καθετήρας διπλού αυλού**
για εντερική σίτιση (CH 8) και αποσυμπίεση (CH 16) **Μόνο για εντερική χορήγηση.**
- es - **Sonda de doble lumen**
para alimentación intestinal (CH 8) descompresión gástrica (CH 16). **Solo para uso enteral.**
- fi - **Kaksilumeninen letku**
intestinaali ruokintaan (CH 8) ja gastrisen paineen poistoon (CH 16). **Ainoastaan enteraaliseen ravitsemukseen.**
- hr - **Duplolumenska sonda**
za intestinalno prehranu (CH 8) gastrične dekompresije (CH 16). **Samo za enteralnu upotrebu.**
- hu - **Dupla lumenű szonda**
intesztinális tápláláshoz (CH 8) gasztrális dekompresszióhoz (CH 16). **Csak enterális alkalmazásra.**
- is - **Tvöföld holslanga**
fyrir næringaræði og í meltingarveg (FR 8), þrýstingsmínkun maga (FR 16). **Aðeins til notkunar í meltingarvegi.**

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2. Indications

This special tube is intended to be used for intestinal feeding of critically ill patients suffering from: gastric stasis; gastric emptying disorders; pyloric stenosis. In addition the tube is also suitable for essential diagnostics and treatments on patients in intensive care, e.g. drainage of gastric fluids; pH measurement of gastric juice; early diagnosis and treatment of gastric bleeding.

3. Contraindications

Nasopharyngeal stenosis; severe injuries to the nasal and pharyngeal cavities, severe clotting disorders.
Important: As with all feeding tubes, the Freka® EasyIn may cause irritation of the mucous membranes during or after placement. In isolated cases this may result in gastrointestinal bleeding. The physician treating the patient should consider the risks associated with placement of the Freka® EasyIn.

4. Placement Procedure

Positioning the intestinal tube through an endoscopic instrument channel ≥ 2.8 mm

- Check whether the size of the tube FR 8 is compatible with the instrument channel of the endoscope.
- Insert the endoscope orally and advance it as far as possible through the stomach towards the intestine (Fig. 1).
- Before introducing the intestinal probe into the operating channel of the endoscope provide both the probe and the operating channel with water or MCT Oil.
- Once the desired position has been reached, advance the intestinal tube through the instrument channel of the endoscope to reach under observation a position behind the ligament of Treitz (Fig. 2). Sometimes a guidewire can be used to assist achieving this position.
- Carefully withdraw the endoscope. Hold the tube in position by using a reverse motion (Fig. 3).

Re-routing the tube from oral to nasal position

- Moisten the blue re-routing catheter with a local anaesthetic gel and carefully introduce it through the nose and out through the mouth (Fig. 4).
- Now insert the intestinal tube through the opening of the re-routing catheter which exits the patient's mouth, and withdraw it via the nose (Fig. 4). Ensure that the tube is in position against the rear wall of the pharynx without any loops.
- Remove the re-routing catheter. The intestinal tube is now in a transnasal position (Fig. 5).

Positioning the gastric tube

- Before installing, moisten the probe with water or MCT Oil.
- Advance the gastric tube, tip first, over the intestinal tube as far as the desired position in the stomach (Fig. 6), making sure that the intestinal tube remains in position.

Connecting the adaptors

- Cut off the intestinal tube approx. 3 - 4 cm beyond the funnel-connector from the gastric tube (Fig. 7) and advance the Click-connector, with the grey end first, over the intestinal tube (Fig. 8).
- Insert the metal pin of the ENFit connector as far as possible into the intestinal tube (Fig. 9) and screw the Click-connector tightly up to the stop (Fig. 10). **The metal pin of the ENFit connector has to be clean and free of grease.** To prevent accidental opening of the Freka® Click-adaptor, the screw cannot be opened without destroying it.
- Advance the Freka® Click-adaptor onto the Funnel-connector of the gastric tube until a click is clearly heard (Fig. 11). The intestinal tube must not shift its position significantly when the adaptor is fixed.

Securing the tube

- Record the marking on the tube at the level of the nostril in the tube protocol. Use the plaster supplied to secure the tube on the bridge of the nose and cheek.

X-ray verification

- The correct position of the intestinal tube has to be verified via x-ray. The tube has to be free of tension and straight, i.e. without loops, in a position well behind the ligament of Treitz.

5. Decompression

Decompression is carried out via the gastric lumen of the Freka® funnel-connector (white, marked "g" for gastric) either using gravity or by active aspiration.

6. Duration of use

The Freka® EasyIn can remain in position for up to 4 weeks, if the tube and the nose are carefully cared for. If enteral feeding is required after this time, introducing a percutaneous endoscopic gastrostomy (PEG) is recommended.

7. Care of the tube

Both before and after feeding applications / gastric decompression, the jejunal tube lumen and the gastric decompression lumen has to be flushed with 20 ml cooled boiled water; if not in use, at least once per day.

Important advice: No acidic liquids, in particular fruit teas or fruit juices, may be given, as they can cause food parts to separate, causing the tube to occlude. If the tube occludes it has to be replaced.

The lumen of the tube must never be cleared by applying force, e.g. high pressure (via small syringe) or a guide wire. Otherwise there is a danger of damaging the tube and injuring the patient..

8. Application of medicines

Liquid-Medicines should be administered if possible. The tube has to be flushed carefully before and after each application. **Medication must never be administered with food.** If in doubt, consult the pharmacist.

9. Tube Feeding

Feeding into the small intestine can begin as soon as the tube is in place. For jejunal tube feeding, high molecular weight tube formulas or low molecular weight oligopeptide formulas may be used, appropriate to the patient's ability to digest and absorb. Feeding has to be increased gradually, and continually controlled by a enteral feeding pump. Infusion pumps, which are designed for parenteral application, must not be used (risk of confusion)!

10. Notes on reuse of the device

This product is for single use only. Reuse creates a potential risk of patient or user infection. Contamination of the device may lead to injury, illness or death of the patient. Reprocessing may compromise the structural integrity of the device. For further information see www.fresenius-kabi.com.

11. Ordering Information

Product name	piece	Product code
Freka® EasyIn, ENFit	1x1	7981833
Freka® Plaster for securing tube	1x50	7981907

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